


Support and reinforce soft tissue *and*
minimize soft tissue attachments (STAs)


**Don't try this with
anything but SurgiWrap®.**



Through a trocar

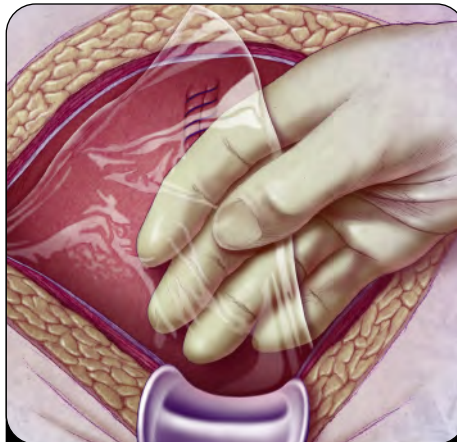

Obstetrics

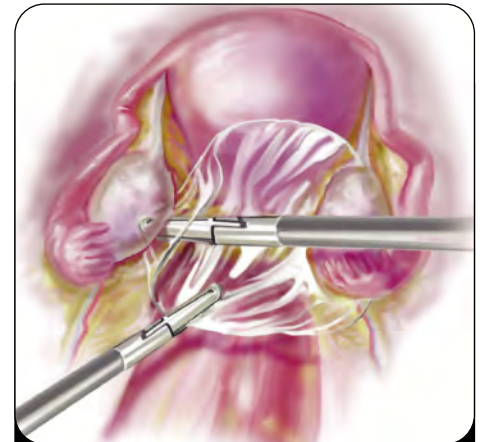

Laparoscopy

**The first and only bioresorbable protective sheet
indicated for open or laparoscopic surgery**

Cleared by the FDA

- Safe and effective in all soft tissues of pelvic and abdominal cavities

 **SurgiWrap** MAST

Bioresorbable Protective Sheet

It's in your hands.

2 studies document the safety and efficacy of SurgiWrap[®]1,2

In a prospective pilot clinical study with 47 gynecologic patients, laparoscopically delivered SurgiWrap[®] (0.02mm) resulted in:

- Successful placement in all patients
- No adverse events in a 5-month follow-up period


After 37 re-looks in 354 patients over a 4-year period who had SurgiWrap[®] placed during C-sections:

- Only 3 patients were found to have STAs in the area where SurgiWrap[®] was placed
- There was a low incidence of adverse events (1.23%) possibly related to SurgiWrap[®]
- In 134 cases between September 2005 and November 2006, there were no reported adverse events or return visits to the ER

It's in your hands

Placing SurgiWrap[®] is quick, easy and intuitive

After introduction into the sterile field, simply:

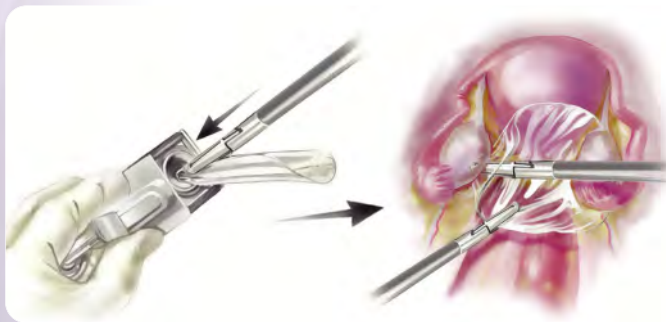

1. Roll lengthwise while holding sheet with laparoscopic graspers and deliver through camera port
2. Place on operative site and reposition as desired
3. Secure with fine sutures or clips or anchor in place with anatomy

References: 1. Data on file. (Shahmohamady B, Saberi N, LaShay N, et al. To evaluate the feasibility of laparoscopy application of a polylactic acid (SurgiWrap[®] MAST[®]) bioabsorbable sheet.) 2. Lamoutte H, Chatterji R. *SurgiWrap[®] MAST[®] Bioresorbable Sheet Use for the Prevention of Soft Tissue Attachment; A Two Year Experience.* Clinical Data Series in Bioresorbable Technology. Available at www.mastbio.com

SurgiWrap MAST[®]

Bioresorbable Protective Sheet

It's in your hands.



MAST Biosurgery Inc.
6749 Top Gun Street, Suite 108, San Diego, CA 92121
Phone (858) 550-8050 Fax (858) 550-8060
Visit us online at www.mastbio.com

©2007 MAST Biosurgery Inc. All rights reserved. Printed in the USA.
US Patent No. 6,531,146. International Patents Pending.
All trademarks and registered trademarks are the property of their respective owners.

LIT 114_B